Hiti - Reusable Warmers Round - RU20100

EN

APPLICATION:

- The reusable Hiti was developed by specialists, like athletes, trainers, physiotherapists and doctors. It makes a local precise treatment of, pulled muscles, arthritis, backache and hypothermia possible.
- The Hiti does not need any electricity or preheating. Its deep heat can be used everywhere.
- By pressing the button the Hiti heats up to 55°C within seconds. The durability is almost twice as long as of typical 'Gel' packs. Being used carefully, the Hiti can be reused for hundreds of applications.

ACTIVATION:

1. Take the metal disc **(1)** using thumb and forefinger.
2. Quickly bend the disc **(1)** until it clicks.

REGENERATION:

Use boiling water.

1. Wrap Hiti in a towel and then put it in a pot in a way that it does not get in touch with the pot. Then boil for 15 - 20 minutes.
2. Take out Hiti and check on solid parts. If there still are solid parts, shake it for about 15 seconds and put it back into the boiling water for 3 minutes. Then it will have the status before use.

WARNING:

1. Do not use it straight after the regeneration. It is very hot and causes burns.
2. Supervise use of Hiti by children, invalids or persons with sensitive skin.
3. Do not open the Magic Hiti. If the Hiti inside leaks out, do not allow it to contact the skin or enter the eyes.

SAFETY REMARK:

Do not put it directly on the skin of persons having circulatory problems or over sensitivities. First ask a doctor. Keep it out of reach of children.

© 2009 Rubytec. All rights reserved. Rubytec® is a registered brand name of Robijns B.V.

While we have taken great care to ensure that information contained in this documentation is both comprehensive and accurate, no warranty of accuracy is expressed or implied. Its content is subject to change at any time without notice.

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© 2009 Rubytec. Tous droits réservés. Rubytec® est une marque déposée par Robijns B.V.